THE DENTAL
SOLUTIONS
COMPANY™

Treatment Centers

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Materials and
general instructions

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Design, technology and quality – an integrated unit

Dentsply Sirona is committed to maximize user benefits. For this reason, it is essential that your treatment center supports the dental workflow consistently so you can work efficiently and ergonomically.

It is the goal that you can focus entirely on your work. For better, safer, faster dental care.

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Design

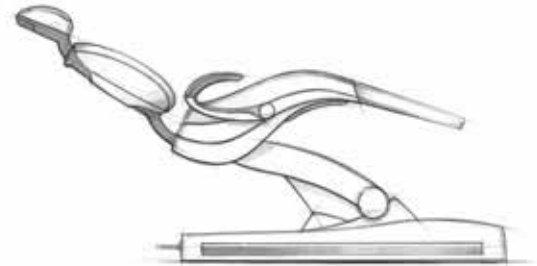
Your commitment to the highest level of quality and aesthetics expresses itself directly in the design of our treatment centres. This is manifested in the perfect interaction between form and function, coupled with outstanding stability. Modern design features streamline the user's workflow and make a positive impact on the comfort and well-being of the patient. Your commitment is our mission to be among the best in the industry. Our proof are the awards that our treatment centers have won throughout the years: we are proud that we have won the product design award, the iF design award and the iF communication design award several times.

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Materials

We value high quality and longevity of our materials. We combine tried-and-tested materials and new alternatives. This results in a level of quality that you can see and feel. We ensure the highest quality standards, durability, stability and aesthetics - right down to the very last detail. A further decisive advantage is that our treatment centres are "Made in Germany". Development, production and quality assurance all take place at our factory in Bensheim.

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General instructions – Care and cleaning

This brochure contains important information about materials, their special properties, as well as care and cleaning recommendations. These recommendations supplement, but do not replace, the information contained in our operating instructions. Further details can be found in the document “Care and Cleaning by the Practice Team“. A continuously updated list of approved detergents, disinfectants and care products can be downloaded from the Dentsply Sirona website (Service > Care and cleaning).

National hygiene and disinfection requirements/guidelines must be observed (e.g. RKI, ADAS, CDC, etc.).

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General instructions

To ensure long-term user satisfaction your Dentsply Sirona treatment center requires regular cleaning, care and disinfection. Special recommendations apply.

Please pay attention to the following:

- Avoid any contact with rough surfaces and sharp objects.
- The earlier contamination is detected, the easier it will be to remove.
- Stains and medicinal product residues must be removed immediately.
- You should use only those care and cleaning products that have been recommended by Sirona.
- Please pay attention to detailed information contained in the operating instructions.

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Upholstery

Properties

The look and feel of the upholstery depends on a smart combination of materials. The surface is waterproof and thus hygienic. The durability of the upholstery plays a major role in maintaining the softness and ergonomic shape of the seat cushion and backrest. Our upholstery materials are selected on the basis of clinical and functional criteria and then processed and assembled using perfect craft skills. Subject to regular care and cleaning, the upholstery will retain its original sheen and suppleness.

Basic factors:

- Light-coloured upholstery is more prone to dirt accumulation than darker coloured equivalents.
- Patients should never be seated on damp upholstery.
- The upholstery should always be thoroughly dried.

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Paintwork colors

Properties

The surfaces of our high-quality products are either spray-painted or powder-coated. Despite their inherent hardness, the paintwork surfaces are prone to wear and tear. For this reason all contact with rough and sharp objects should be avoided.

Care and cleaning

- Remove any dirt and disinfectant residues with the aid of a mild detergent and then wipe dry.
- Do not clean and disinfect the paintwork surfaces with the aid of dyed wipes. Remove medicinal product residues immediately using a damp cloth.

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Glass

Properties

The glass cuspidor bowl creates a light, translucent impression. Fingerprints remain invisible due to the bowl's frosted surface. Although glass is a relatively robust material, it is nonetheless prone to scratches. For this reason avoid any contact with rough surfaces, sand grains and sharp metal objects.

Care and cleaning

- Please observe the operating instructions when cleaning the monitor, the operating lights and the glass cuspidor bowl.

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Plastics

Properties

Plastics adapt themselves to a wide range of different requirements. Despite their low weight, they are elastic, robust and durable. Plastics are pleasant to the touch, resistant to slipping and do not cause allergies. Above all, they are easy to clean and disinfect.

Care and cleaning

- Remove any dirt and disinfectant residues with the aid of a mild detergent and then wipe dry.
- Do not clean and disinfect the plastic surfaces with the aid of dyed wipes.
- Remove medicinal product residues immediately using a damp cloth.

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Stainless steel and Melamine

Properties

Stainless steel is a high-strength material which is corrosion-resistant and easy to clean. Enamel is a durable vitreous material which is resistant to chemicals and medicinal products. In the case of the cuspidor bowl an enamel coating is applied to a steel surface. As is the case with glass materials avoid any contact with rough surfaces, sand grains and sharp metal objects.

Care and cleaning

- Remove any dirt and disinfectant residues with the aid of a mild detergent and then wipe dry.
- Clean the enamel cuspidor bowl in accordance with the operating instructions.

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Sirona Dental Systems GmbH